

PLASTER SEPARATOR / -TRAP

Item No. | 10001

with sealed, 4-chamber clarification system

› no unpleasant odors › no spillage › easy to clean

Transparent container with plastic sheet inlay and snap-shut locking system. Transparency of plaster separator enables visual monitoring of sludge. Plastic bag inlay ensures easy and rapid cleaning.

European-Patent No. 2627426

TECHNICAL DATA

Height, diameter:	320 mm, 340 mm
Discharge height:	245 mm
Discharge connector:	Ø 50 mm
Capacity:	14 l
Inlet connection:	1 1/2", Ø 28 mm
Weight:	2,1 kg



Position inlet in the upper chamber.



For cleaning detach the connection of the drain, unscrew closure cap, remove the remaining water with our drawing and pull out the insert.




Remove drain adapter.



Bind the foil inlay under the outlet opening and remove. Temperature max. 80°C

20004 | Replacement inlay plastic bags, set of 25, for plaster separator / -trap (image not shown)

20202 | Discharge connector for double sink 120 mm – 600 mm / 1 1/2"

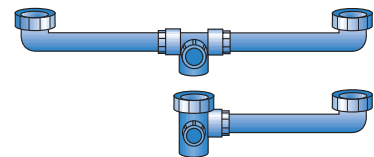

60086 | Discharge connector for double sink 80 mm – 300 mm / 1 1/2"




70073 | Plaster separator supply pipe, straight, 1 1/2"



70074 | Plaster separator supply pipe, 90°, 1 1/2"



90027 | T-connector for double water supply

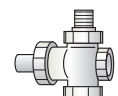

70115 | Flexible discharge hose, L: 1,000 mm, Ø 50 mm, Conn.: 40/50 mm



70125 | Flexible discharge hose, L: 750 mm, Ø 50 mm, Conn.: 40/50 mm



70124 | Flexible discharge hose, L: 500 mm, Ø 50 mm, Conn.: 40/50 mm



WATER EXHAUST PUMP

Item No. | 10027

for plaster separator

The practical complement to the plaster separator for the uncomplicated and quick removal of the residual water.


Remove the cover of the plaster separator. Submerge the pumping tube into the residual water and hang the flexible hose into a separate container. The up and down movement of the pump lever easily and effectively drains the water through the pump, via the hose, to the separate container. The plaster is disposed of in the usual manner.

TECHNICAL DATA

Overall length:	530 mm
Stroke length:	135 mm
Displacement volume:	100 ml/stroke

Aluminium design

Utility Model No. 2020110014673
Community Design No. 001804931-0001

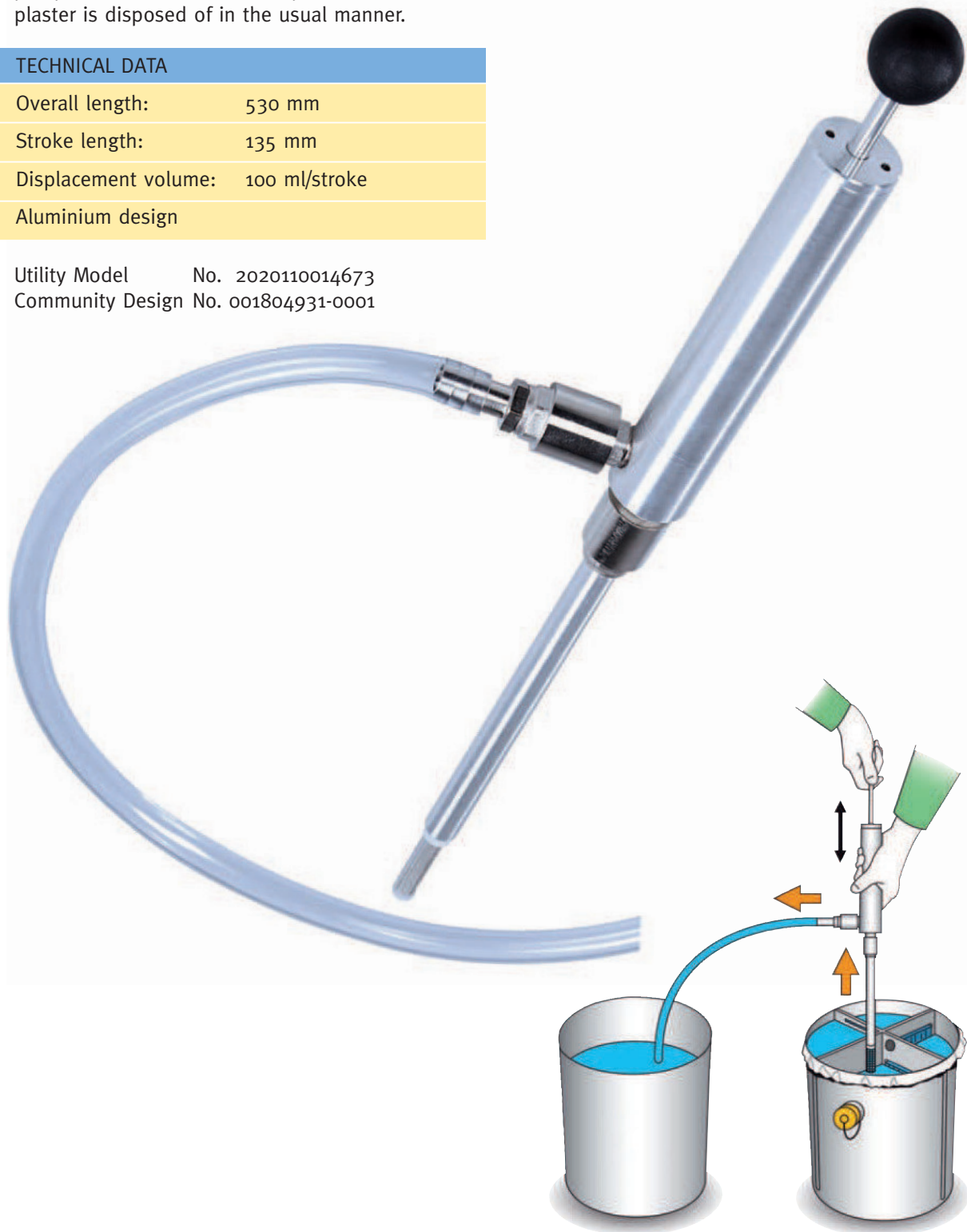

Clots, coagulates and odour free

DentaFlocTabs eliminates 99% of all bacteria and therefore no unpleasant odours occur. The flocculent properties in the DentaFlocTab means over 90% of gypsum are embedded and settled in the plaster separator.

The dispensing basket with the DentaFlocTab needs to be hung in the highest chamber of the plaster separator. At least once a week put a new DentaFlocTab in the dispensing basket, this will prevent odour and maintain efficient coagulation of the plaster.

PACKAGING UNIT

Art.-Nr. 21002 10 pieces

Art.-Nr. 21003 25 pieces

