SINTERSCHALE SiC

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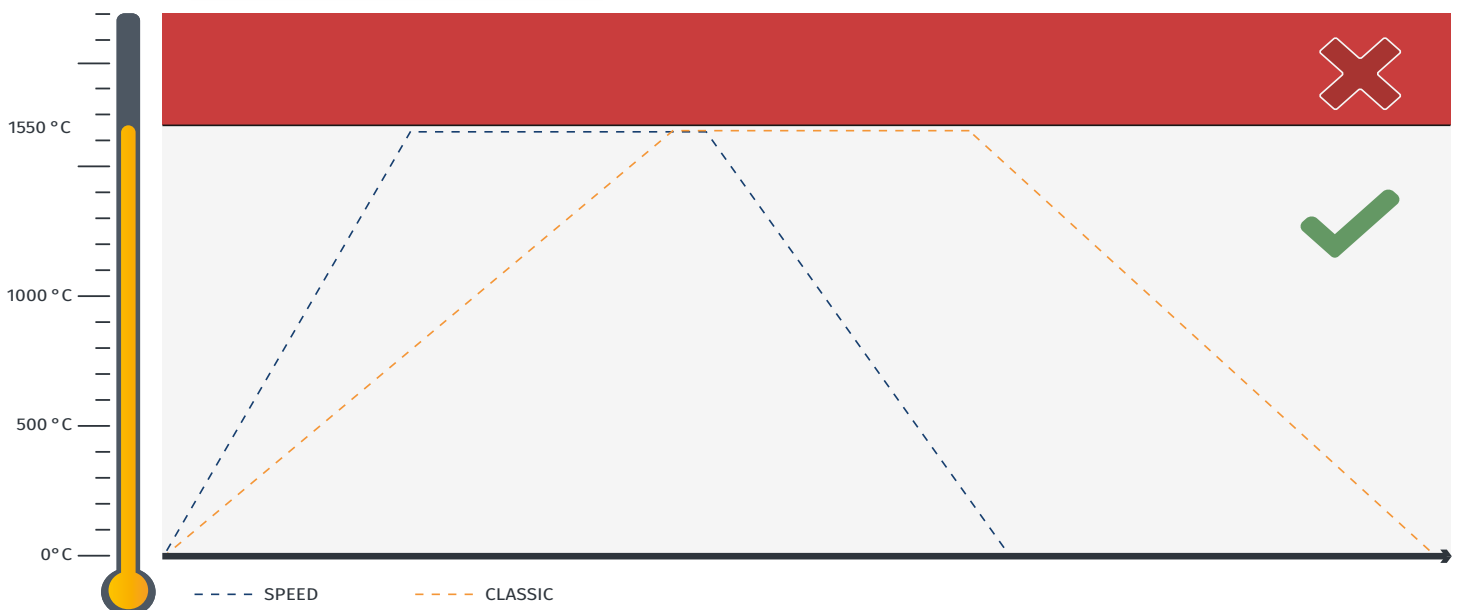
SINTERING TRAY SiC

- ▼ Maximum temperature 1550°C
- ▼ Sintering tray on material basis SiC
- ▼ For SPEED and CLASSIC processes
- ▼ High thermal shock resistance
- ▼ Long service life
- ▼ No discoloration of the work

**Höchsttemperatur
von 1550 °C beachten!**

**Observe the maximum
temperature of 1550 °C!**

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PLEASE OBSERVE THE WEAR OF THE THERMAL FABRIC SUPPORT AND
REPLACE IT, IF IT NO LONGER ADHERES TO THE TRAY.

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COMPARISON OF SINTERING TRAYS DURING OPERATION LIFE

This sequence shows the regular modification in appearance of our new and innovative SiC sintering tray, when used according to our instructions at temperatures up to 1550°C. The longevity of the material has been proven in various test cycles, with CLASSIC as well as SPEED sintering programs. The sintering tray can be used either with sintering pearls or granulate, always providing excellent results without any negative impact on zircon quality.

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TRACES OF USE

With proper use, and increased operating life, the material is subject to slight discoloration, resulting from constant exposure to insulation dust.

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SLIGHT OVERHEATING

Overheating of the sintering tray will cause light blistering of the material. Reprocessing of the tray is usually possible according to instructions (see instruction sheet enclosed).

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SEVERE OVERHEATING

Intense overheating of the sintering tray will lead to strong blistering. The material structure has suffered lasting damage, which may cause the tray to break. Reprocessing of the tray is no longer possible.

Systemgröße 100 | system size 100

Best.-Nr. / Item No.	Bezeichnung	Description
8203000032	Auflagevlies, 100; für Sinterschale (Satz-5)	Thermal fabric support, 100; for sintering tray (set-5)
8201000048	Sinterschale SiC, 100/30	Sintering tray SiC, 100/30
8202000017	Tiegelzange 18/10, 100; gebogen	Crucible tongs 18/10, 100; bent

Systemgröße 120 | system size 120

Best.-Nr. / Item No.	Bezeichnung	Description
8203000033	Auflagevlies, 120; für Sinterschale (Satz-5)	Thermal fabric support, 120; for sintering tray (set-5)
8201000047	Sinterschale SiC, 120/30	Sintering tray SiC, 120/30
8202000018	Tiegelzange 18/10, 120; gebogen	Crucible tongs 18/10, 120; bent

Supported & Distributed By:

Atlanta, GA - (770) 921-2131
805 Marathon Parkway
Suite 130
Lawrenceville, GA 30046
8am- 5pm EST. Monday- Friday

Dallas, TX - (972) 600-9187
16135 Preston Rd.
Suite 126
Dallas, TX 75248
8am- 5pm CST. Monday- Friday

Website: www.zublerusa.com

Email Us: sales@zublerusa.com

Status: 02/21 Revisionsnum: 2.07.2021
This document was created with
Free changes and errors excepted. Figures and figures similar.

MIHM-VOGT GmbH & Co. KG
Friedrich-List-Straße 8
76297 Stutensee-Blankenloch
Germany

Web www.mihm-vogt.de
Mail info@mihm-vogt.de