


VARIOstar

Micromotor

Spindle replacement




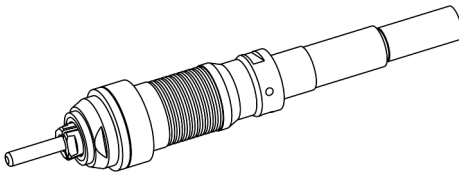


en

2. Maintenance

2.3 Replacement of the spindle

2.3.1 Removal of the spindle

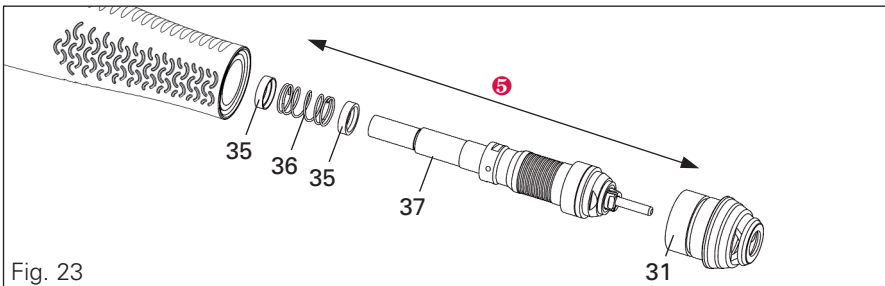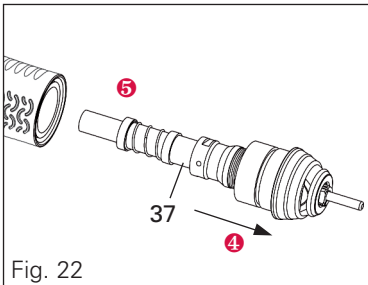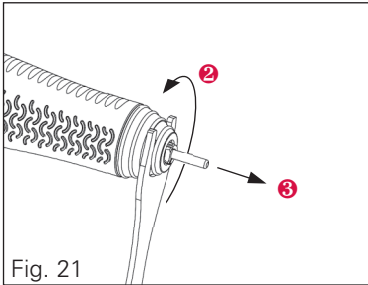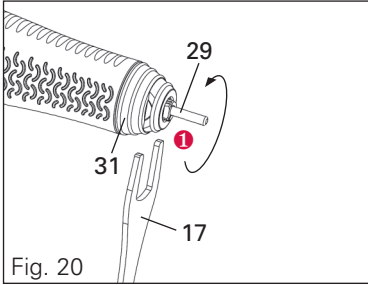


1 Clamp pin **29** or a milling bit.

Turn the pin **29** until the flanks of the spindle **37** are visible at the tip **31** and the flat spanner **17** can be inserted.


2 Insert the flat spanner **17** and completely unscrew the tip **31** in a counter-clockwise direction.

3 Pull out the tip **31** with the spindle **37**

4 Pull the tip **31** off the spindle **37** and insert the new spindle **37**.

5  Make sure that the two spring holders **35** and the spring **36** are pushed onto the new spindle **37**. Make sure that the shoulder of each spring holder **35** is facing the spring **36**.

6 Now push the spindle **37** with the tip **31** back into the handpiece and screw it in.

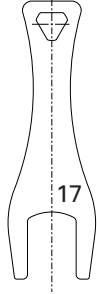



M-VARIOstar 38/2015en
Subject to technical changes!!

 **zubler**[®]
dental & technik

www.zubler.de



Zubler Gerätebau GmbH
Buchbrunnenweg 26
D-89081 Ulm-Jungingen