


Sintering on fleece (thermal fabric) can be a very good alternative to sintering dishes & trays for speed sintering or if you find you are cracking a trays or dishes excessively. Fleece can cope with any heating rate and cooling rate due to it's lack of rigidity allowing it to handle thermal expansion and contraction much better than harder, more dense materials.

Das Sintern auf Vlies ist eine sehr gute Alternative zu den Schalen. Denn das Vlies kann nicht brechen oder Risse bekommen, wie bei den Schalen. Das Vlies macht jede Aufheizrate und Abkühlrate ohne Problem mit.

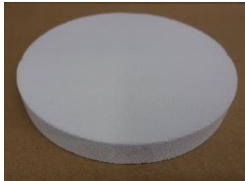

Support Disc diameter 120mm
Auflagescheibe Durchmesser 120
mm Part number: 8203000002 (Each)



Fleece diameter 120mm
Vlies Durchmesser 120 mm
Part number: 8203000033 (Pkg of 5)



Sintering Granulate 200g
Sintergranulat 200 g
Part number: 8201000004

Recomended Usage:

The fleece is placed on top of the support disc and granulate is spread over the fleece, leaving the edge free. Restorations may then be placed on the granulate. Optimally, the granulate and fleece remain on the support disc in the furnace and zirconia units are removed one by one.

Das Vlies wird auf die Auflagescheibe gelegt und dann das Granulat auf das Vlies verteilt, der Rand sollte freibleiben. Danach können die Restaurationen auf das Granulate gelegt werden. Um es optimal zu verwenden, sollte das Vlies auf der Platte im Ofen verbleiben und die Zähne einzeln herausgenommen werden

